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
Digital alcohol breath tester „BLACK/WHITE”

INSTRUCTIONS FOR USE

1. Clock LCD
2. Alcohol tester LCD
3. Power button
4. Breath inhaled
5. MODE button
6. SET button
7. ADJ button
8. Battery compartment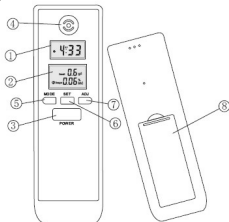


Features

1. Dual LCD display.
2. Digital alcohol test with audible alert.
3. Range: 0.00-0.19% BAC & 0.0-1.9g/l.
4. Quick response and resume
5. Auto power off.
6. Real time & alarm timer.
7. Temperature.
8. Count down timer.
9. Use 2× "AAA" alkaline batteries.

Installing battery


1. Slide open the battery compartment lid from the back of the unit.
2. Insert two "AAA" (UM-4) alkaline batteries with correct polarity as indicated.
3. Replace the battery compartment lid.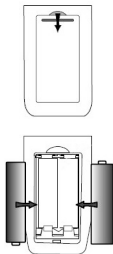

How to use the alcohol tester

Warm up

1. Press and hold the power button for 1 second, the alcohol tester LCD will turn on with one beep sound.
2. The "warm up" symbol is displayed, and the 10 to 00 countdown is started that indicate the tester is in warm-up mode.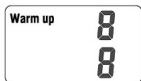


Note:

It always has a large number of gas & impurity adsorbed on the surface of alcohol tester, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and warm up the sensor, you should repress the power button and retry several times if possible.

Test

1. The "Blow" symbol is displayed when warm-up is succeed, and the 10 to 00 countdown is started that indicate the tester is in testing mode.

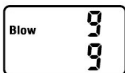2. Blow into the breath inhaler for 3-5 seconds with normal power after a deep inhalation..
3. Read the test result from the alcohol tester LCD.
4. The "Caution" symbol is displayed if the alcohol concentration is the level:
 - 0.02% BAC (or 0.2g/l) to 0.05% BAC (or 0.5g/l)
 - 0.05% BAC (or 0.5g/l) to 0.08% BAC (or 0.8g/l)

5. The "Danger" symbol is displayed with beep audible alert if the alcohol concentration is the level:
= > 0,05% BAC (or 0,5g/l).
6. If no blow into the breath inhaler at the test mode, the LCD will display "0.000 %BAC & 0.0g/l" value.

Note To obtain more accurate result, manufacturer recommend you to check repeatedly 2 or 3 times.

Auto power off

After the result be display for about 10 seconds, the "OFF" symbol will appear for 2-3 seconds and the alcohol tester be turned off.

Replace battery

LCD screen flashes for low power, please replace the alkaline batteries.

How to use the clock

Real time

Press <ADJ> button to select 12 or 24 hour format. Press and hold <SET> button for 2 seconds to enter real time hour setting. When the hour digit is flashed on the clock LCD, press <ADJ> button to adjust. Press <SET> button again to enter the minute setting. When the minute digit is flashing, press <ADJ> button to adjust. Press <SET> button to confirm.

Alarm time

Press <MODE> button once to enter alarm mode. Press <ADJ> button to select alarm on or off. Press and hold <SET> button for 2 seconds to enter alarm timer hour setting. the hour digit is flashed, press <ADJ> button to adjust. Press <SET> button again to enter the minute setting. the minute digit is flashed, press <ADJ> to adjust. Press <SET> button to confirm.

Temperature

Press <MODE> button twice to enter temperature mode. Press <ADJ> button to select C (Celsius) or F (Fahrenheit) format.

Count down timer

Press <MODE> button 3 times to enter timer mode. Press and hold <SET> button for 2 seconds to enter the timer hour setting. When the hour digit is flashing, press <ADJ> button to adjust. Press <SET> button again to enter the minute setting. the minute digit flashed, press the <ADJ> to adjust. Press <SET> button to confirm. Press <ADJ> button to start the countdown. When the timer value is not 00:00, press <ADJ> button again to stop the timer. When the timer counts down to 00:00, the alarm will sound., then reload the timer value.

Backlight

when the tester is turned on, the alcohol tester LCD backlight will light until tester auto power off.

Specifications

Sensor	semiconductor oxide alcohol sensor
Range	0.00%-0.19% BAC (0.0-1.9g/l)
Accuracy:	0.01% BAC (0.1g/l)
Power input	2x 1,5 V AAA battery (<i>not included</i>)
Operating time	warm up: 10s, response: <5s
Clock accuracy	+/- 2.0 seconds/day
Auto power off	1 minute approx
Model	01 901

Precaution

- When after drinking, it is recommended to last after 20 minutes, it is because 20 minutes is approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat next testing within 3 minutes after last time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly.
- Do not blowing liquid into the breath pipe.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Modifying or tampering with the unit' s internal components may cause a malfunction.

Regardless of whether the product is used, manufacturers and vendors will not bear any responsibility for the violations. For example: drunk driving and high-altitude operations.

Each person reacts differently to alcohol, so the test results are for reference only, which cannot serve as the criteria in determining the outcome.

Instructions and information on management of used packaging

Dispose of used packaging material to a designated municipal waste facility.

Disposal of used electrical and electronic equipment

This symbol on products or accompanying documents means that used electrical and electronic products should not be treated as household waste. For proper disposal and recycling, take these products to designated collection points. Alternatively, in some European Union states or other European countries you may return your products to your local retailer when buying an equivalent new product. Correct disposal of this product will help save valuable natural resources and prevent any potential negative impacts on the environment and human health, which could otherwise be caused by improper waste disposal. For more details, please contact your local authorities or the nearest collection site. The improper disposal of this waste can be in accordance with national regulations for fines.

**For business entities in the European Union**

To dispose of an electrical or electronic device, request the necessary information from your seller or supplier.

Disposal in other countries outside the European Union

This symbol is valid in the European Union. To dispose of this product, request the necessary information about the correct disposal method from local government departments or from your seller.

This product meets all the basic requirements of EU directives that apply to it.

Changes to the text, design and technical specifications are subject to change without prior notice and we reserve the right to change them.

Compass CZ s.r.o. Pražská 17,
250 81 - Nehvizdy, CZ, e-mail: info@compass.cz

revize textu 2/2017